# Meni-LAB 0,5%

Professional use

## INDICATIONS

Specific care regimen for RGD GAS PERMEABLE TEST LENSES  
Disinfection solution with an application procedure which eliminates UTA  
(Unconventional transmissible agents).  
This solution is intended for professional use.

## CHARACTERISTICS

Meni-LAB 0,5% is a product for professional use specifically for use with RGP trial lenses.  
Meni-LAB 0,5% solution was developed to disinfect RGP trial lenses against:  
- bacteria, yeasts and moulds  
- viruses  
- amoebae  
and depending on the procedure recommended, eliminate Unconventional Transmissible Agents.  
Meni-LAB 0,5% must be used after each test with the original lens container.  
Soaking time for lenses in the Meni-LAB 0,5% solution is 5 minutes.  
Meni-LAB is also a cleanser (Menicon internal data).

## INSTRUCTIONS FOR USE

- On removing the lens, place it in its storage bottle filled with MeniCare Pure or MeniCare Plus. Then clean the lens with MeniCare Pure or MeniCare Plus solution, rubbing for 20 seconds. Then rinse with MeniCare Pure or MeniCare Plus solution.
- Insert the lens into the storage bottle holder, empty the storage bottle then clean the lens as detailed in the following instructions:
- Open the container of Meni-LAB 0,5% carefully and fill the original lens container with the solution up to the neck.
- Screw the lens holder onto the container. Leave the lens to soak in Meni-LAB 0,5% solution for 5 minutes to disinfection stage.
- After 5 minutes, unscrew the lens holder and empty the contents of the container. Rinse the lens carefully for 15 seconds with MeniCare Pure or MeniCare Plus solution.
- Rinse the lens holder and lens container too.
- Fill the lens container with MeniCare Pure or MeniCare Plus solution.
- Screw the lens holder onto the container and save the lens like this until the next test.
- In the event of extensive soaking, renew the soaking solution every weeks.
- Before fitting, rinse the lens with MeniCare Pure or MeniCare Plus solution.
- NB: every treatment must be recorded (traceability document)

## PRECAUTIONS

- Do not use for soft lenses.**  
The soaking time for trial lenses in Meni-LAB 0,5% solution must not exceed one hour. Longer than that, the quality will not be affected but the lenses may be discoloured.
- Important: do not swallow. Do not leave within reach of children.**  
It is essential to rinse the lenses abundantly with MeniCare Pure or MeniCare Plus solution after soaking them in Meni-LAB 0,5% solution.
- Do not put the solution directly in your eye. In the event of accidental splashing,**  
rinse abundantly immediately. Avoid contact with the skin and do not splash onto clothing. Store at room temperature (15-25°C). Close the container properly after use.
- Use within 3 months after opening.
- Respect the expiry date noted on the container.
- Never use tap water to rinse your lenses.
- 25 ml container
- © 2017, Menicon Pharma SAS

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- Active chlorine solution ..... qs. 0,5% or 5 000 ppm of active chlorine
- Sodium hydroxide
- Purified water ..... ..... qs 100%

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