GB Spray & Clean

INDICATIONS

Spray & Clean is a preservative free solution indicated for cleaning of all rigid gas permeable contact lenses. The specific device ensures sterility of the solution during use.

DIRECTIONS

Before each contact lens manipulation, wash your hands.

Remove respectively your lenses from the eye into the palm of your hand.

Spray one shot of solution on each side of the lens.

Rub the lens carefully in your palm with your fingers or between fingers.

Put the lens on related lens holder of the lens case and rinse the lens thoroughly with MeniCare Plus or another rinsing solution.

Fill your lens case with MeniCare Plus solution and screw the cap.

Frequency of use: can be used every day. Follow the instructions given by your contact lens specialist.

Before wearing lenses:

Rinse the lenses with MeniCare Plus or adapted rinsing solution. The lens can be worn.

Throw away the remaining storage solution from lens case.

WARNINGS

Keep out of children's reach.

PRECAUTIONS

• Validity after initial use : 3 months.

• Do not touch dispenser tip.

• Store at room temperature. 

• Observe the expiry date printed on the label.

 • Important : do not swallow or drink.

 • Do not spray in the nose.

• Do not put Spray & Clean directly into your eye.

• Use MeniCare Plus or your regular for rinsing.

• Store out of direct sunlight. 

CONTRA INDICATIONS

• Do not use for soft lenses.

• Do not use in case of allergic background to one of ingredients. In case of problems contact your eye care practitioner.

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