

Cantor&Nissel

C U S T O M • L A T H E C U T

ChromaGen™ Practitioner Assessment Forms



ChromaGen™ is a Cantor and Nissel product

Form 1

Evaluation Stage One- Part 1

Please indicate if any of the symptoms listed below apply. You must answer all questions and indicate "N" ONLY if it does not apply in full. If you believe that it tends to occur, then please circle "Y".

Circle as appropriate;

- | | | |
|--|---|---|
| 1. Difficulties in following verbal/written instructions; | Y | N |
| 2. Is there any hearing problems; | Y | N |
| 3. Difficulties in telling time (reading clock) | Y | N |
| 4. Difficulties in finding the right word to express themselves | Y | N |
| 5. Poor hand/eye co-ordination (find handwriting/parallel lines) | Y | N |
| 6. Short attention span/easily distracted | Y | N |
| 7. Tendency to reverse letters (eb d/b, p/q) or miswrite (s/z, v/x, n/h) | Y | N |
| 8. Poor reading skills | Y | N |
| 9. Word reversals (e.g. Was/Saw) | Y | N |
| 10. Reluctant to read aloud or reading to others | Y | N |
| 11. Poor homework and presentation of written work | Y | N |
| 12. Great efforts and labour in completing homework or simple tasks | Y | N |
| 13. Poor or no organisational skills | Y | N |
| 14. Difficulties in copying words from books/blackboard/bad temper | Y | N |
| 15. Becomes frustrated with tasks leading to tantrums/bad temper | Y | N |
| 16. Poor self esteem due to literacy difficulties | Y | N |
| 17. "It is unfair" and "Why me?" attitude because others do better | Y | N |
| 18. Poor/gross mistakes with simple words | Y | N |
| 19. Good at oral communications to ideas, reluctance to put in writing | Y | N |
| 20. Good at practical skills e.g. with tools, DIY, drawing | Y | N |
| 21. Poor short term memory or poor working mental memory (math) | Y | N |
| 22. Difficulties with sequence logic or number/works relationships | Y | N |
| 23. Difficulties with place value/ratio value/fractions value | Y | N |
| 24. Difficulties in estimating answers/use of finger counting | Y | N |
| 25. Misreads familiar words e.g. Home=House, Mother=Mum | Y | N |
| 26. Reluctance to have a go at reading difficult words or simple tasks | Y | N |
| 27. Drifts away form looking at texts when reading/loses reading place | Y | N |
| 28. Little or no expression and intonation with reading | Y | N |
| 29. Doesn't recognise the same word on next page or else where in story | Y | N |
| 30. Uses finger with reading, loses patients, irritable with reading | Y | N |
| 31. Attempts to spell phonetically | Y | N |
| 32. Difficulties in distinguishing vowel sounds form vowel combinations | Y | N |
| 33. Poor use of capital letters and punctuation, capital letters in mid word | Y | N |
| 34. Messy handwriting to hide poor spelling, leading to dislike of writing | Y | N |
| 35. Reluctance to ask for help or being helped by parents | Y | N |
| 36. Negativism and bad temper when faced with even simple school tasks | Y | N |
| 37. Whining and sibling rivalry | Y | N |
| 38. Cannot read what he/she wrote | Y | N |
| 39. Dictation is fair, but copying is poor to impossible | Y | N |
| 40. Word blindness, misses out words altogether and can skip a line | Y | N |

Score with any allowance given to Age: -


5yrs old - 5 points	6yrs old - 4 points	7yrs old - 3 points	8yrs old - 2 points	9yrs old - 1 point
Score 1 point For each Applicable statement	No of points Scored less Allowance = points	Total no of Points;	Total score Out of 40: /40	Ratio (MAX 1) SLD stage Anticipated x 100 = %

Please enter any additional information or comments:


Form 2

Referral


Investigation & Assessment

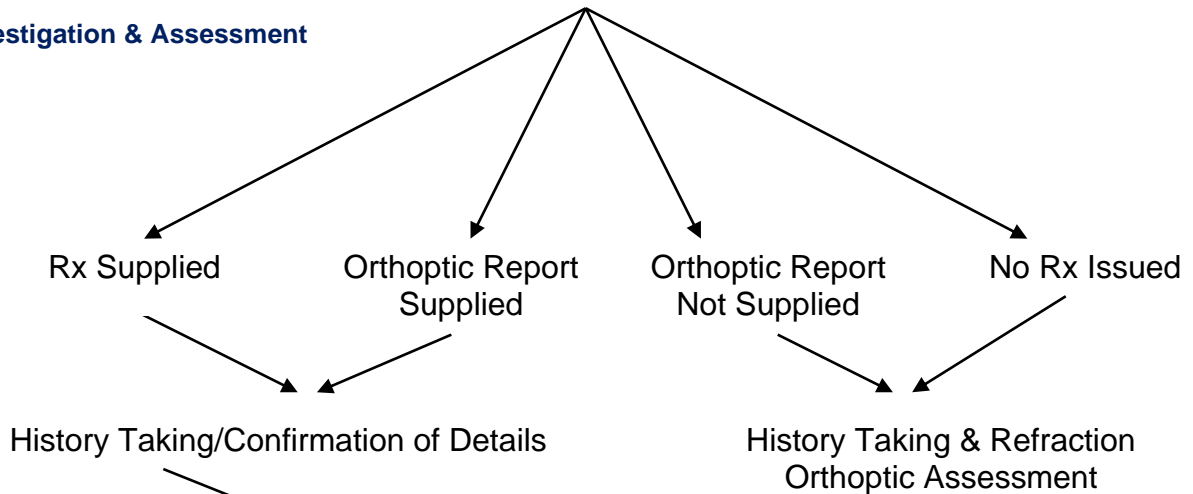

Evaluation & Pre screening

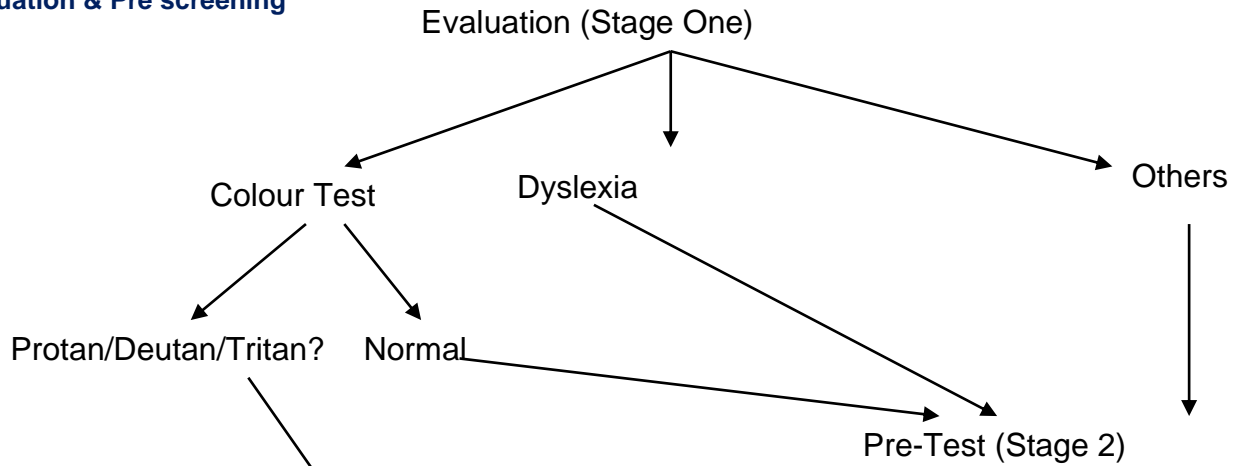

Dispensing & Summary

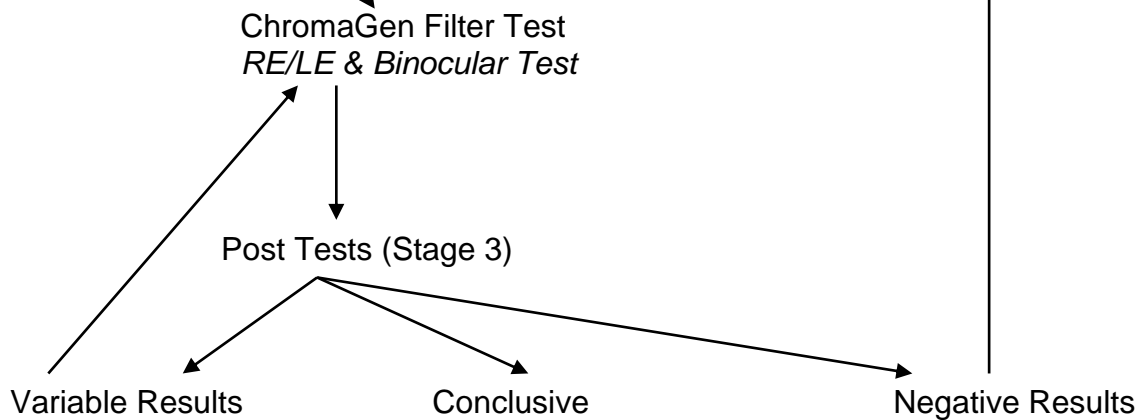

Recommendation

Dispense Filter & Review in 6 Weeks

Dispense filter Review in 3/12

From 2a

Academic Skills Disorders (Dyslexia) Appraisal Test

Name:	DOB:	CL/Spx/Filter:	Date:
--------------	-------------	-----------------------	--------------

Optical Prescriptions:

	Unaided VA	Sphere	Cyl	Axis	Prism	Base	Base VA	Near Add	N	Interm	IOP
RE:											
LE:											

Test Performed	Without Filters	With Filter (State Which)	Comments
Colour Test: 1. Ishihara 2. City Test 3. D15			
Stereoscopic: 1. TNO 2. Fly			
Maclure Bar Reading Test			
NV Test Charts: 1. Log NV 2. Bail-Lovie			
Contrast Sensitivity 1. Vistech 2. Pelli-Robs			
Performance: 1. Numbers 2. Circles 3. Reading 4. Crowding			

Cantor & Nissel

C U S T O M · L A T H E C U T

ChromaGen™ Practitioner Assessment Forms

**These sheets should be photocopied for use in
assessments.**



ChromaGen™ is a Cantor and Nissel product

Single Word Reading

Please read as many words as possible in one minute starting from group 5 onto 15+

5	8	12	15+	Special allowance
<p>In on to dad door cat let to bee hop and door mum to on me dad ant cat dog as the air to see the dog and was out old ran to in the day dog</p>	<p>Three from after and work this were some test there man what be from were this work and this year take from three there and this test work this three and after take from what man said</p>	<p>Show win you that his had show cry many before and read three any near suit their before and many post and read them there hours show the same from before suit and their well hours cry show win some</p>	<p>Lead has come school find mean their design often number ball home face read before jump measure their design line another loss school measure their lead date call near home come and see mean date measure design has come</p>	<p>I to dad to door at the door bee hop dog cat to was air</p>
<p>Words read from this section:</p>	<p>Words read from this section :</p>	<p>Words read from this section:</p>	<p>Words read from this section:</p>	<p>Words read from this section:</p>
<p>Number of words missed</p>	<p>Number of words missed</p>	<p>Number or words missed</p>	<p>Number of words missed</p>	<p>Number or words missed</p>
<p>How many words can you read from ALL the sections:</p>				
<p>How long did it take you to read all the words:</p>				
<p>List down the words you found difficult to read:</p>				
<p>This section is for our use. Please do not write in this section</p>				
<p>Chron. Age _____ Perform. Age _____ Perform % _____ Level _____</p>				
<p>Notes:</p>				

SINGLE WORD READING

5-7 year old	8-11 year old	12-15 year old	Over 15 yrs	Special allowance
hop cat and car ant eat in a not man on us i all to my the dog me we in up air do see for not look is play come let be dad was in by at old out	date back dog my use over eye who was six for the late house call and as out man what be there some test lead date use work this were and from have then year take from after and three	Spell type was report were then may date have won you win that his above work that due three any near read show above when day post hand show cry many before from their suit well house find them there	Cause known vision either certain optical result often the mirror type be yet tried place loss test ball face read was before number home jump measure their design line loss another school find mean near date call lead has come	in l on to me dad at door ant cat dog let to hop bee
Time taken to read the randomised words:				
Accuracy performance:				

Overall Score:

Read as many words as possible from the words listed below in one minute:

*man one
 see ear use
 win come now
 even in zoo saw
 race coin swan our
 mess nose six cow sin
 we rise car sun oar rise
 scan miss wise mare roar
 on river sea omen mace cave
 even swim see nice sir crow zone
 maze wine raw cress rain rum
 cure bed sum sine use seam man
 air arm cove sky pig ball go bat*

Words read per minute: /60

SENTENCE READING TEST:

Here is a man and here is a boy. The boy has a dog. See the dog in the car.	The man has a dog the man and the boy like the dog. The boy is in the car
4-6	
The children like to go to the farm to see animals. Cow, dog, cat, fish and pig	They like to play with the horses and feed the lambs, pigs, cats and fish
6-7	
Next day at the school, the teacher placed a picture and asked the children to look at it	The children tell the teacher their ideas for the picture with many trees and fish in the pond
7-8	
After school the children go home for tea. At first Peter wanted to do a puzzle. He quickly picked the pieces up and sat On the floor to play.	Slowly or quickly the game may change depends on which way Peter and Jane wanted to swing. They had an idea that made the game more fun.
8-9	
Uncle Ted was happy to communicate with the children about his journey abroad. He wanted to surprise them, so he did not tell them about the journey planned for them today.	They were surprised and delighted to know that they were going up in a helicopter. The search for the best part of today was now over. They always wondered how to fly an aeroplane
9-10	
Soon they were flying over an island. The helicopter went suddenly down and disappeared into the woods. As it approached the sand it created a large cloud of dust. Every thing around them seems to disappear for a while	They watched silently as the helicopter landed safely and then quickly they jumped out on the beach. "Shall we go to the top of the cliff?" The pilot said. Look there, a disappearing figure seems to run into the woods.
10-11	
Now the children were delighted with their holiday, they wondered if they can go back to the old building and discuss their exciting trip with the old man who lived in the castle. He refused to speak to them first but soon learned they were friends and meant no harm.	The children took a present for him and searched for some more interesting items to give the old man, They searched and found an old panel, It had a secret code and when eventually they opened it, they found it decorated with unusual writing and strange patterns.
11+	

Chron. Age:

Reading Age:

Words read in Error	Words Inserted	Words Missed	Words altered

Form 6

A

d l a p m s e r z o n f w t b

g c h i x j q u k v y m h r e

did be of ate i to ten in s t my x

by if e he go up o p we m as if it

good camp fire our fear less gust hot

always cowboy funny shells weaver night

blueberry flowers eyesight instrument

red great slender fix truth awful k no

seesaw stale louder score radish overly

postcard automation rainbow devilish

hand some midshipman hamper minnow tuner

frighten rabbit mustard crab grass difference

completion d cast butter start scar homeland

B

x g a j p m u l c d s b r h o

f w z i t b k e n q v y a m r

yes so j pop sat d at c am h in so

oh of n to am g k in u do of s b

fire side past gold fish own sky help

advice badger slide anyone table mirror

understudy sportsman campground fenders

bas z navy specific g show dog amber

narrow today penny cream hopped honest

meantime upbringing summertime splendid

quick sand spiteful outlast stops winds


side walk tracking readily overshadow employ

story milk bunny college crayons idea gotten

Form 7

Please copy the following shapes *five times each* on the lines:



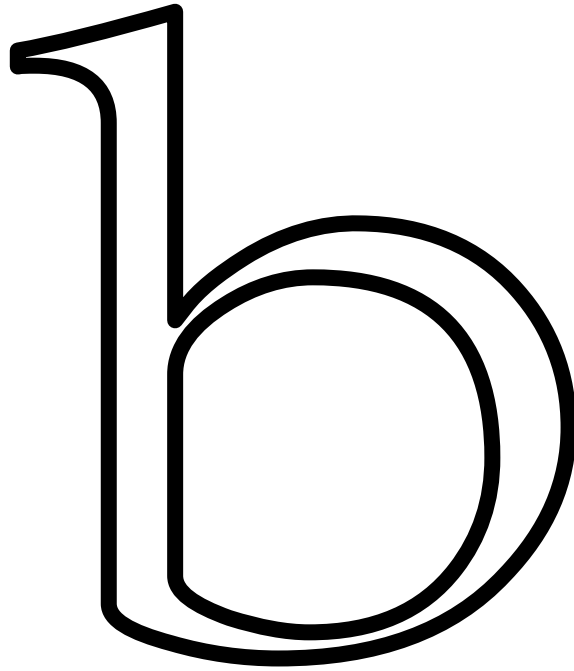


rsbrfgtbdaspl tqabdstyghbk bdfrmsbdswpyltbhd

fgthbsaderfh bdt dtflhpmonfbdestyphklsmdbwfrty

hjtbfhkbds pbhtyqjishytbsrybpludfhvbkdpfrbdo

bsrftklpomufgbs drtyfhds wabc moprbdjholtrsploh

kfklptomhgbd wasdrwqpfklbsthyopgbsfderttsma

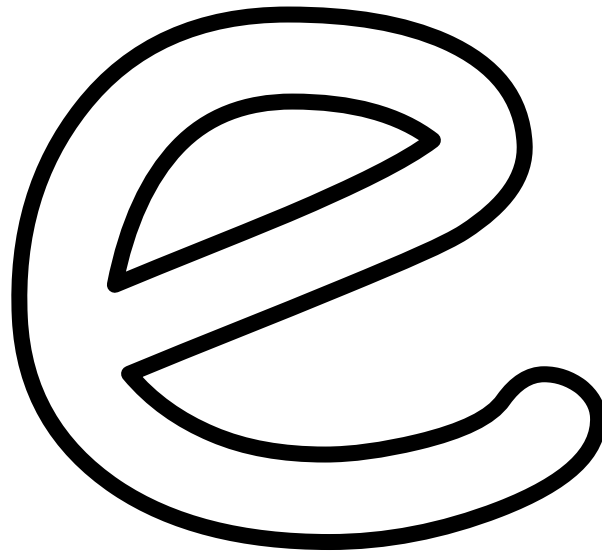

rdsaecesnmedaeocaopcesteaoacanzrweocoidmasesea
esaeceomnuiuoersftwqseaoiuedceafrocoimnmotaecaf
oiunoedraeacoeongjuoieasevevorghyopsacaonoincc
faceonouoiunomeisaxzeoinmomfgewsaelpomonuoej
xoiuinboiencodgedbasaeceonomfcbcuieouacemong

6

8576436807263562096806720853102061470611993205
7369087695463711067094736100326742285510609426
30650963210993160060413885360743260204518690324
4730954967138604592054120607032983152058364221
5218536092160963052544026182940327148729630216
3072056830546982014327320504976821732060438271
2856201578293475608060986731253284271047203152
9246608217032968427691002104284922002001670924

6

8576436807263562096806720853102061

7369087695463711067094736100326742

3065096321099310060413885360743260

4730954967138604592054120607032983

5218536092160963052544026182940327

9

5309214798205628944210953781974205790639
6029435044092193063281395748252059301026
03597569604206910932750802538162069032470
49519289723749102769010936890839205891572
10930792740965209154260492063738204601912
72965109274927008912437926102259327104697
02692731405790283648191203090275387752179
16302837492730571829579308069832053416893

9

53092147982056289442109537819

60294350440921930632813957482

03597569604206910932750802538

49519289723749102769010936890

10930792740965209154260492063

Form 14

A

6 8 3 1 0 9 9 5 2 7 4 0 2 7 9

7 1 7 0 2 8 6 3 2 0 1 6 9 3 6

20 45 78 12 8 57 9 40 6 37 11 5 71 6 29 4 24

18 35 76 89 98 6 54 67 4 57 20 85 2 7 29 76 18

37 28 95 87 26 53 84 92 61 69 83 61 48 72 59 73 26 51 68 77

83 76 227 18 332 948 76 996 21 679 82 443 710
82 338 27 294 447 729 53 892 146 692 29 47 294 37 601
6835 5398 6386 3859 4062 7403 4861 2850 4621 6429 65485

B

8 4 9 5 3 1 6 0 4 2 6 8 4 7 5

5 2 8 9 4 2 1 6 4 9 6 7 3 5 2

42 5 56 89 32 478 578 63 42 4 67 8944 623 17

35 866 42 36 97 846 31 256 798 45 3 10 15 7 88
48 99 62 47 62 64 19 82 77 36 18 44 72 39 67 18 20

21 44 675 7 55 890 3 34 20199 888 14 2244 1161 543
77 452 97 908 335 23 787 115 22 787 49 1 685 856 88 643 446 12
2456 3790 9786 5444 2740 6202 1901 7998 3454 1143 6765

VERSION A

Come see the play look up is cat not my and dog for you to the cat up dog and is play come you see for not to look my you for the and not see my play come is look dog cat to up dog to you and play cat up is my not come for the look see play come see cat not look dog is my up the for to and you to not cat for look is my and up come play you see the dog my play see to for you is the look up cat not dog come and look to for my come play the dog see you not cat up and is up come look for the not dog cat you to see is and my play is you dog for not cat my look come and up to play see the

VERSION B

See the look dog and not is you come up to my for cat play not up play my is dog you come look for see and to the cat look up come and is my cat not dog you see for to play the my you is look the dog play see not come and to cat for up for the to and you cat is look up my not dog play see come you look see and play to the is cat not come for my up dog come not to play look the and dog see is cat up you for my and is for dog come see the cat up look you play my not to dog you cat to and play for not come up the see look my is the come to up cat my see dog you not look is play and for

VERSION C

You is not come cat up look to play dog my the for and see look for up and the play is dog cat not see to come my you to not play come see look cat you for up the my is dog and the look you dog is cat play see come to for and up not my cat up to my the see and come dog for is you not look play see and dog come cat is play the not look to up my you for dog to cat the is you my see for up play come and not look up my for and to come dog see look cat you is the play not come you is see my for look to not the dog play cat and up is the look to cat not and come play for you up my see dog

VERSION D

Cat for the you not up my dog see to is come play look and see you dog for is cat look play my the up and come not to look cat see my and dog the is play come not for to you up my see is the come play look for and up to cat you dog not and you cat look to see not my dog the for come is up play dog to and play up come you the not is cat look for my see the play look cat see up come for my you and not is to dog for not to you come play the look cat see is dog up and my you dog and for up not see my cat is the play come to look come not and to see you is play look up the cat dog my for

ChromaGen™ Assessment Forms – ASD

Name: _____ D.O.B: _____

Symptoms: _____

DOMINANT EYE: _____

VERSION A – Without ChromaGen filters

Come see the play look up is cat not my and dog for you to the cat up dog and is play come you see
1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25
for not to look my you for the and not see my play come is look dog cat to up dog to you and play cat
26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43 44 45 46 47 48 49 50 51
up is my not come for the look see play come see cat not look dog is my up the for to and you to not
52 53 54 55 56 57 58 59 60 61 62 63 64 65 66 67 68 69 70 71 72 73 74 75 76 77
cat for look is my and up come play you see the dog my play see to for you is the look up cat not dog
78 79 80 81 82 83 84 85 86 87 88 89 90 91 92 93 94 95 96 97 98 99 100 101 102 103
come and look to for my come play the dog see you not cat up and is up come look for the not dog cat
104 105 106 107 108 109 110 111 112 113 114 115 116 117 118 119 120 121 122 123 124 125 126 127 128
you to see is and my play is you dog for not cat my look come and up to play see the
129 130 131 132 133 134 135 136 137 138 139 140 141 142 143 144 145 146 147 148 149 150

Missed Words _____ Added Words _____ WPM _____

VERSION B – Without ChromaGen filters

See the look dog and not is you come up to my for cat play not up play my is dog you come look for
1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25
see and to the cat look up come and is my cat not dog you see for to play the my you is look the dog
26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43 44 45 46 47 48 49 50 51
play see not come and to cat for up for the to and you cat is look up my not dog play see come you
52 53 54 55 56 57 58 59 60 61 62 63 64 65 66 67 68 69 70 71 72 73 74 75 76
look see and play to the is cat not come for my up dog come not to play look the and dog see is cat up
77 78 79 80 81 82 83 84 85 86 87 88 89 90 91 92 93 94 95 96 97 98 99 100 101 102
you for my and is for dog come see the cat up look you play my not to dog you cat to and play for not
103 104 105 106 107 108 109 110 111 112 113 114 115 116 117 118 119 120 121 122 123 124 125 126 127 128
come up the see look my is the come to up cat my see dog you not look is play and for
129 130 131 132 133 134 135 136 137 138 139 140 141 142 143 144 145 146 147 148 149 150

Missed Words _____ Added Words _____ WPM _____

Total WPM:
Comments;

VERSION C – With ChromaGen filters R: _____ L: _____

You is not come cat up look to play dog my the for and see look for up and the play
1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21
is dog cat not see to come my you to not play come see look cat you for up the my is
22 23 24 25 26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43
dog and the look you dog is cat play see come to for and up not my cat up to my the
44 45 46 47 48 49 50 51 52 53 54 55 56 57 58 59 60 61 62 63 64 65
see and come dog for is you not look play see and dog come cat is play the not look to
66 67 68 69 70 71 72 73 74 75 76 77 78 79 80 81 82 83 84 85 86
up my you for dog to cat the is you my see for up play come and not look up my for
87 88 89 90 91 92 93 94 95 96 97 98 99 100 101 102 103 104 105 106 107 108
and to come dog see look cat you is the play not come you is see my for look to not
109 110 111 112 113 114 115 116 117 118 119 120 121 122 123 124 125 126 127 128 129
the dog play cat and up is the look to cat not and come play for you up my see dog
130 131 132 133 134 135 136 137 138 139 140 141 142 143 144 145 146 147 148 149 150

Missed Words _____ Added Words _____ WPM _____

VERSION D – With ChromaGen filters R: _____ L: _____

Cat for the you not up my dog see to is come play look and see you dog for is cat look
1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22
play my the up and come not to look cat see my and dog the is play come not for to
23 24 25 26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43
you up my see is the come play look for and up to cat you dog not and you cat look to
44 45 46 47 48 49 50 51 52 53 54 55 56 57 58 59 60 61 62 63 64 65
see not my dog the for come is up play dog to and play up come you the not is cat
66 67 68 69 70 71 72 73 74 75 76 77 78 79 80 81 82 83 84 85 86
look for my see the play look cat see up come for my you and not is to dog for not to
87 88 89 90 91 92 93 94 95 96 97 98 99 100 101 102 103 104 105 106 107 108
you come play the look cat see is dog up and my you dog and for up not see my cat is
109 110 111 112 113 114 115 116 117 118 119 120 121 122 123 124 125 126 127 128 129 130
the play come to look come not and to see you is play look up the cat dog my for
131 132 133 134 135 136 137 138 139 140 141 142 143 144 145 146 147 148 149 150

Missed Words _____ Added Words _____ WPM _____

Total WPM:
Comments:

ChromaGen Filters Prescribed: R: _____ L: _____
Recheck Due: